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ENGLISH

General description and purpose of the medical device:

Arti-BruX® is a red indicator paint, which is applied with a brush (single use product) to the occlusal surfaces of restorative and prosthetic appliances as well as hard splints. After drying, Arti-BruX® Quick forms a thin and even lacquer coating. The lacquer coating is intended for short-term use (up to max. 24 hours). The lacquer coating is saliva proof and is being rubbed off by occlusal contacts. Arti-BruX® Quick shows occlusal contacts and movement traces of the teeth by means of the removed lacquer layer.

For professional use only.

Indication

Arti-BruX® Quick BK 88 is a red liquid contact color:

- To control active dynamic occlusion on restorative and prosthetic restorations as well as on bite splints.
- For short-term control of occlusion (max. 24h) - Occlusogram.
- For visualization of enamel fissures as well as margin fissures of fillings and crowns.

Contraindication

- Arti-BruX® Quick consists mainly of concentrated alcohol (ethanol 96%). Arti-BruX® should not be applied directly invasively to patients who are addicted to alcohol, pregnant women or children. Also contact with injured mucosa should be prevented.
- Arti-BruX® Quick must not be used in case of iodine allergy.
- Arti-BruX® Quick must not be applied to open dentin or prepared teeth.

Side Effects

- With insufficiently polymerized acrylics or vacuum splints, made of different thermoplastics, discoloration of the plastic may occur.
- If the surfaces to be tested are not properly polished or sand blasted, the lacquer can penetrate into micro cracks of the plastic or ceramic surfaces and cause discoloration.
- When Arti-BruX® Quick is applied, the color can penetrate into enamel cracks or margins of fillings and cause temporary or even permanent discoloration.
- After longer use of Arti-BruX® Quick (up to max. 24 hours) discoloration of the saliva may also occur; the lacquer consists of a natural resin and red food coloring. After use, the mouth should be rinsed with water.

Notes

- Arti-BruX® Quick is a single use product and may only be used within one treatment session to prevent the transmission of germs.
- Disposal: the single use applicator and excessive paint must be disposed of with the usual contaminated waste.

Package content

- Single use applicator, content 0,15ml
- Instructions for use

Application

- The surfaces to be tested have to be cleaned and dried thoroughly.
- Apply the paint to the occlusal surfaces to be tested.
- To control bruxism, especially sleep bruxism, the patient should insert the splint shortly before going to bed.

Effect/Diagnosis

Check the surfaces to be tested after the end of the observation period. The coating is rubbed off by tooth contacts (occlusal surfaces). These contact patterns are used to check the occlusal contacts in static and dynamic. A uniform wear, e.g. on bite splints, is an indicator that the individual splint is evenly loaded. Strong one-sided wear indicates parafunctions (e.g. bruxism) or an incorrectly adjusted splint. The same applies to prosthetic restorations.

Cleaning (of the surfaces to be tested)

After successful inspection, remove the coating from the surface. Appropriate tools are: tooth brush and appropriate tooth paste, alcohol bath or wadding soaked with alcohol (Ethanol 96%). Prosthetic restorations can be cleaned with a toothbrush or standard practice equipment for the professional tooth cleaning.

Disposal Instructions

Arti-BruX® Quick consists of a red food color and a natural resin, which is also approved for food and pharmaceutical applications. All components of the kit can be disposed of with the usual practice waste.

Notification of Incidents

Serious adverse events occurring in connection with the product must be reported to the manufacturer and the competent authority of the Member State.

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