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ENGLISH

General description and purpose of the medical device:

Green color paste for marking sore spots on dentures and partial dentures.

For professional use only.

Indication

Bio-Ink® Flow is indicated for marking sore spots on the gum and subsequent marking on the denture.

Sore Spots also known as decubitus can occur frequently with new dentures or partial dentures. Also a degeneration of the alveolar ridge can cause sore spots as well as unbalanced occlusal forces on the denture. The adaption of the oral mucosa to the new denture often causes inflammatory lesions. Bio-Ink® Flow is a highly viscous paste. By punctiform, laminar or linear application to the mucosa the Palatal Vibrating Line and the mucobuccal fold can be marked. If necessary, the denture can be adjusted.

Contraindication

A direct application onto highly inflamed areas or purulent lesions of the oral mucosa areas should be avoided.

Side Effects

Possible side effects could be allergic reactions.

Notes

- The color paste can be applied directly with the disposable cannulas (single use) and spread with the disposable applicator (e.g. Magic-Brush™) to avoid cross-contamination.
- Alternatively, the paste can be used with a single use applicator (e.g. Magic-Brush™) and then applied to the surface to be tested.
- The product must be stored in the sealed syringe at temperatures between 0°C and 40°C.

Package content

- 1 x Bio-Ink® Flow Paste, Syringe 1ml
- 5 single use cannulas
- Instructions for use

Application

Before use make sure that the mucosa, palate and alveolar ridge are dried carefully with a cellulose pulp or a cotton roll so that the color paste does not run. There should be no saliva on the surface to be tested. The denture must also be cleaned and dried carefully beforehand.

Effect/Diagnosis

The color marking of the sore spot will be transferred to the denture when the denture is seated into position. When the denture is removed the spot is visible on the surface of the denture. If necessary, the denture can be adjusted by the dentist.

After control, the color marking can be easily removed with water and a suitable surfactant (dishwashing detergent).

Cleaning (of the surface to be tested)

After control, the color marking can be easily removed with water and a suitable surfactant (dishwashing detergent).

Disposal Instructions

Bio-Ink®-Flow consists of a green food color and natural waxes, which are approved for food, cosmetics and pharmaceutical applications. All components of the kit can be disposed with the usual practice waste.

Notification of Incidents

Serious adverse events occurring in connection with the product must be reported to the manufacturer and the competent authority of the Member State.

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 REF Bestellnummer
Reference number

 LOT Chargen-Nummer
Batch code (Lot number)

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